

MediCRUSH

Home of the *original* mediCRUSH pill crusher



Instructions:

- Lift lever of pill crusher to upright position.
- Insert .5 oz crush cup into crushing bowl.
- Place medication into .5 oz crush cup. Do not over fill the cup. Limit crushing to 2-3 tablets at a time.
- Place second .5 oz crush cup on top of medication. This will prevent cross contamination.
- Apply force to crush, and rotate the crush cup if needed.
- Always consult your pharmacist or physician before crushing any medication.
- Caution: Time released medication should not be crushed.

*If you have any questions, please contact Medi Crush and Supplies Company, LLC at 601.832.7556 or 601.898.3248

