
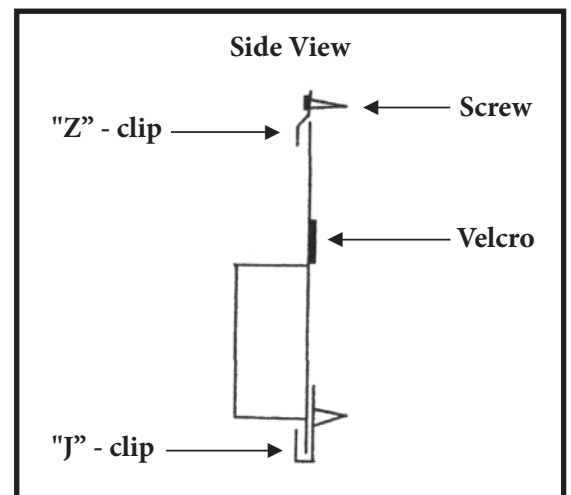
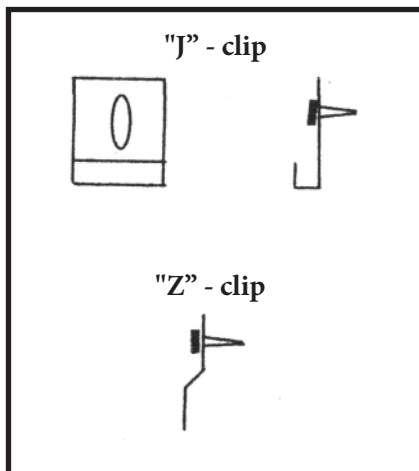


Installation Instructions

PD-1 & 2

1. Place the hook/stiff side of the small Velcro strip on the back of each section as indicated in the drawing below, and then reattach the loop/soft side. Leave the protective film on the loop side for now.
2. Locate the support tubes/studs in the wall (contact ambulance manufacturer) on which you intend to install your organizer.
3. Using a 7/64th drill bit, drill 4 holes (making sure that the holes are level with each other) where you intend to attach the "J" clips. (see diagram below)
4. Attach "J" clips using 4 of the supplied screws.
5. Remove the protective film from the loop side of the Velcro and place the left section in the "J" clips. Have someone hold it in place.
6. Using the 7/64th drill bit, attach the "Z" clips at the top as indicated below.
7. Repeat steps 5 and 6 for the right side.

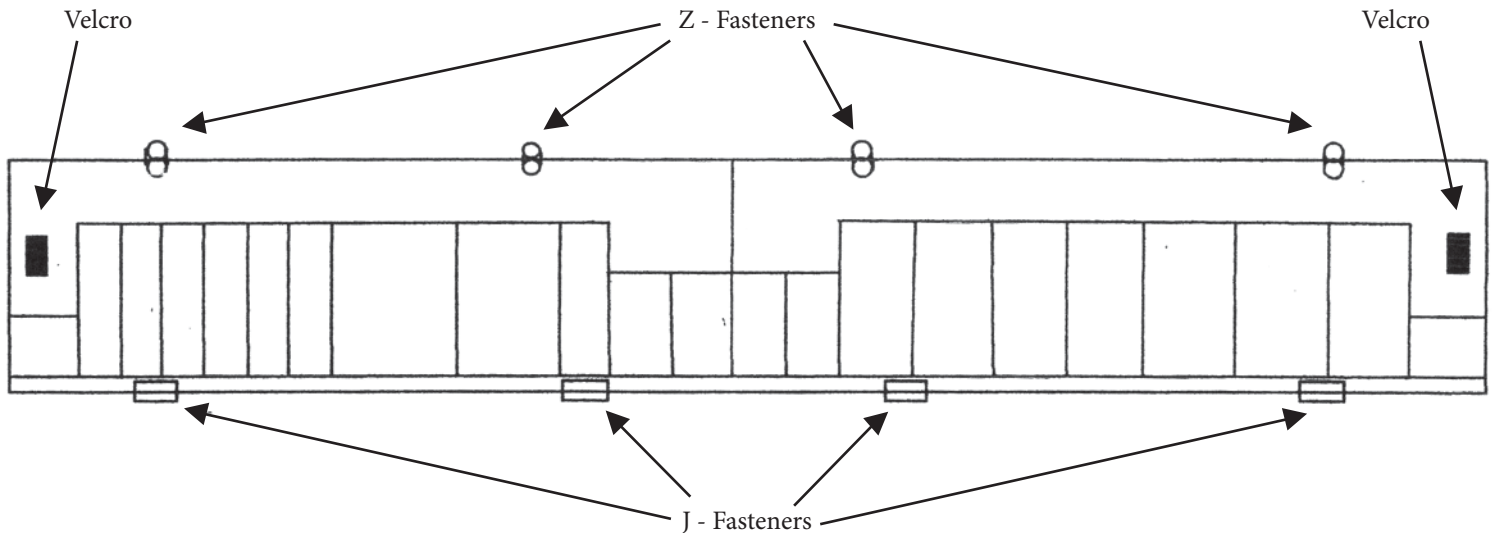

Installation Instructions

PD-3/4/5

1. Place the hook/stiff side of the small Velcro strip on the back of each end as indicated in the drawing below, and then reattach the loop/soft side. Leave the protective film on the loop side for now.
2. Locate the support tubes/studs in the wall (contact ambulance manufacturer) on which you intend to install your organizer.
3. Using a 7/64th drill bit, drill 2 holes (making sure that the holes are level with each other) where you intend to attach the "J" clips. (see diagram below)
4. Attach "J" clips using 2 of the supplied screws.
5. Remove the protective film from the loop side of the Velcro and place the organizer in the "J" clips. Have someone hold it in place.
6. Using the 7/64th drill bit, attach the "Z" clips at the top as indicated below.

